



QUICK REFERENCE GUIDE

Setting up the CuddleBlanket

On

Turn CuddleBlanket on

In

Put the CuddleBlanket Hose into the machine

Sheet

Add your care provider's sheet to the bed, the person can now be laid on the bed

Pad

Add the cooling pad on top of the person
(you can also add an inco sheet here)

Sheet

Add your care provider's sheet to the pad to protect the insulated blanket

Blanket

Add the CuddleBlanket Insulated Blanket
(you can add a sheet or personal blanket on top of blanket or insert inside a duvet cover)

WHEN TURNING OFF

Out

Remove hose first

Off

Turn unit off
