




CuddleCot Quickstart Guide

1. Place insulating foil on surface of cot/Moses basket (silver side facing up).

2. Place cooling pad on top of foil and cover with a thin sheet or insert into pillow case.

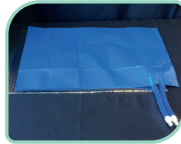

3. Connect the double connectors of hose to cooling mat and the single connector to the cooling unit.



4. Remove filler cap, add two drops of biocide and fill 3/4 with purified water.



5. Plug in the unit and switch on. Check water level and top up if required.

The temperature will read high initially, this will begin to cool over time. Press the downward arrow to ensure the temperature target is 8°C. Then click 'enter' to set this.

6. Cover the baby with blankets and tuck in to aid insulation.

For further instructions and troubleshooting see instructions contained in the CuddleCot box or visit our website

www.CuddleCot.com/manuals

